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 **PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM**

Elastic natural latex band 5 m - 30 m

DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares, under its sole responsibility, that this is a class I medical device that has been manufactured and assessed, according to the EU Regulation 2017/745 (MDR). These instructions have been drawn up in application of the aforementioned Regulation. They are intended to ensure an adequate and safe use of the medical device.

SAFETY PRECAUTIONS

Prior to use, check the band for cuts, small cracks or holes, as these might lead to a sudden tearing of the band. If the band is damaged, dispose of the product and replace it with a new one.

WARNINGS

The Duraband for cardio/strength and rehabilitation exercises may cause serious injury if used incorrectly. We recommend to use the band only on recommendation of a physician or a professional person, and to follow the illustrated instructions. The Duraband must not be used for persons under the age of 8 and, in any case, only under the supervision of an adult. This product contains natural latex and can cause allergies. Do not swallow.

SELECTION/SIZE

Code	PB6001 – PB6006 (5m)		
PB 6001	yellow	light	0,15 mm x 5 m
PB 6002	orange	moderate	0,20 mm x 5 m
PB 6003	green	medium	0,25 mm x 5 m
PB 6004	pink	strong	0,30 mm x 5 m
PB 6005	purple	super strong	0,40 mm x 5 m
PB 6006	silver	extra strong	0,45 mm x 5 m

Code	PB6008 – PB6013 (30m)		
PB 6008	yellow	light	0,15 mm x 30 m
PB 6009	orange	moderate	0,20 mm x 30 m
PB 6010	green	medium	0,25 mm x 30 m
PB 6011	pink	strong	0,30 mm x 30 m
PB 6012	purple	super strong	0,40 mm x 30 m
PB 6013	silver	extra strong	0,45 mm x 30 m

Colour: 6 different resistances (colours)
Option: Pair of handles for Duraband (REF.PB6007)

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RESISTANCE VALUES

The table with the resistance values is useful to have a reference point prior to starting the training. It helps to calculate the resistance required to correctly perform the exercise, in relation to the stretch percentage of the band. For example, if you stretch a band with an initial length of 30 cm to 60 cm, the elongation/stretch percentage is 100%, and the force required for this elongation is 1.12 kg for the yellow band (light) and 6.81 kg for the grey band (extra strong). Tip: The band can be converted into a loop with a simple knot. The value of a 30 cm long loop doubles to the value indicated in the table. After the exercise, the knot can be simply undone.

MAINTENANCE

Store the band at room temperature in a box, in a dark place and protected from sunlight. Keep the band away from sharp objects, be careful with long fingernails, and remove any rings prior to use. Rinse the band under running water and then leave to dry naturally. After the band has dried completely, dust it with talcum powder.

INDICATIONS

- Post-surgical rehabilitation

CONTRAINDICATIONS

- Currently no known

CHARACTERISTICS AND MATERIALS

- Elastic natural latex band for resistance and functional rehabilitation

1 PAIR HANDLES FOR DURABAND (REF. PB6007)

Using the handles, which have a special fixing system, the exercises can be performed in a more efficient and safer manner. Simply guide the Duraband rubber band through the first and second fixing opening and then pull tight.

Extension en %	Résistance en kg					
	Légère	Modérée	Moyenne	Forte	Super-forte	Extraforte
66%	1,02	1,58	1,94	2,83	3,39	5,46
100%	1,12	2,04	2,27	3,39	4,08	6,81
133%	1,35	2,50	2,73	4,08	4,54	7,88
166%	1,58	2,96	3,19	4,64	5,10	9,08
200%	1,81	3,39	3,62	5,46	5,89	10,43

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